

MiniSun – IDEEA Information

Overview of the IDEEA Operation

Following are the eight steps in purchasing and using the IDEEA Recorder.

1. **Register** - All new users of the MiniSun web site need to first register.
2. **Login** - All users must Login to access IDEEA Operations.
3. **Purchase** - Purchase IDEEA devices and MiniSun analysis and reporting.
4. **Start IDEEA** - Start the IDEEA physical activity recording.
5. **Load Data** - Load IDEEA physical activity data onto your PC.
6. **Process** - Process IDEEA data to allow viewing by Tracer.
7. **Upload** - Upload IDEEA data for analysis and reporting.
8. **Report** - IDEEA physical activity reports.

The IDEEA Operations Manual provides detailed instructions concerning how to complete each of these steps. The IDEEA Operations Manual is available on-line after you purchase an IDEEA Recorder

1. Register

All new users of the MiniSun web site need to first register before they can Login to IDEEA Operations. To register enter your First Name, Last Name, E-mail Address, Login Name, Password, Phone Number, Address, City, State, Zip Code and Company Name. Required information on this screen is marked with an asterisk. The Login Name you enter needs to be unique. Two different users may not have the same Login Name. Please remember the Login Name and Password that you enter on the Registration screen. The Login Name and Password entered on the Registration screen need to be entered each time you Login to IDEEA Operations. It is important to enter your E-mail Address correctly because it is used by MiniSun to contact you.

Information entered on the Registration web page can be changed on the Profile web page. The Profile web page is only accessible after you Login to IDEEA Operations.

After successfully entering the Registration information the Confirmation web page will be displayed. If a duplicate Login Name was selected you will be directed to go back to the Registration web page and select another Login Name. If there are no errors in the registration information then you will be directed to the Login web page.

All information you enter on the registration page is kept in complete confidence. Please see MiniSun's Privacy Policy for further information.

2.Login (to IDEEA Operations)

All users need login in order to access the IDEEA Operations web pages. The User ID and Password that were entered on the Registration Screen need to be entered on the Login screen. If you have forgotten your Login or Password then select e-mail MiniSun

MiniSun – IDEEA Information

from the bottom of the page and request that your Login information be sent to you by e-mail.

After correctly entering your User ID and Password on the Login page the IDEEA operation page will be displayed.

3.Purchase

A set of IDEEA sensors and Recorder need to be purchased from MiniSun in order to record physical activity (PA) data. Once you have collected physical activity data, you can analyze the data on your computer or you can purchase MiniSun Reports. When you purchase MiniSun reporting, MiniSun will process your data, provide you with consultation, compare your results with the MiniSun database and create reports for you to download. Purchase options and pricing information are available from the Purchase web page. The MiniSun Purchase web page is only available once you Register and Login to IDEEA Operations. The Purchase web page supports on-line credit card processing. Please contact MiniSun concerning non-credit card purchases.

4.Start IDEEA

Start IDEEA is the first step in using IDEEA. IDEEA can record up to two days of physical activity. Following is an overview of the Start IDEEA process.

Physical Activity Monitoring Process (PA Monitoring Process)

1. The subject puts on the IDEEA sensors and the IDEEA Recorder.
2. Connect the IDEEA Recorder to your PC.
3. Using the special web browser provided by MiniSun select Start IDEEA from the IDEEA menu. Follow the instructions on this page to initiate a physical activity recording. The computer does not need to be connected to the internet for the Start IDEEA program to work.
4. Disconnect the IDEEA Recorder from your PC.
5. Keep the IDEEA recorder running till instructed or up to 48 hours.
6. After the recording is complete the subject removes the IDEEA sensors and IDEEA Recorder.

Detailed instructions concerning the Start IDEEA process and how to use the Start IDEEA web page are available in the IDEEA Operation Manual. This manual is available after you purchase the IDEEA Recorder.

5.Load Data

After IDEEA physical activity data has been collected then it needs to be transferred to your PC for processing. IDEEA physical activity data is collected using the Start IDEEA process described above. Following are the steps required to transfer the physical activity data to your PC.

MiniSun – IDEEA Information

1. Connect the IDEEA Recorder to your PC.
2. Login to IDEEA Operations and select the Load web page.
3. Follow the instructions on the Load web page to initiate the transfer of the physical activity data to your PC.
4. Disconnect the IDEEA Recorder from your PC.

Detailed instructions concerning the Load process and how to use the Load web page are available in the IDEEA Operation Manual. This manual is available after you purchase an IDEEA Recorder.

6.Process Data

To process the data use the special web browser provided by MiniSun. From the IDEEA Menu select the Process Data option. Follow the instructions provided by the Process Data dialog boxes. After completing the Process Data step you can use the Tracer software provided by MiniSun to view the IDEEA Recorder data you collected.

Detailed instructions concerning how to Process Data and how to use Tracer are available in the IDEEA Operation Manual. This manual is available after you purchase an IDEEA Recorder.

7.Upload

The Upload web page is used to transfer physical activity data to MiniSun for analysis and reporting. Physical activity data is captured using the Start IDEEA process described above. Once MiniSun has completed the analysis of the physical activity data your reports will be available on the Report web page.

Detailed instructions concerning the Upload process and how to use the Upload web page are available in the IDEEA Operation Manual. This manual is available after you purchase an IDEEA Recorder.

8.Report

After the physical activity data has been uploaded, MiniSun will analyze the data, provide consultation if required, compare your results to the MiniSun database, and create detailed reports about the physical activity. The Report web page is used to download completed reports created for you by MiniSun. Only reports for data that you uploaded will be available on the Report web page. You can not access other peoples reports and they can not access your reports.

Detailed instructions concerning the Report process and how to use the Report web page are available in the IDEEA Operation Manual. This manual is available after you purchase an IDEEA Recorder.